NUTRICIA  
flocare<sup>®</sup> Infinity<sup>™</sup> III

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# SETTING UP THE FLOCARE® INFINITY™ III FEEDING PUMP

AT FIRST PUMP START-UP AND AFTER A FACTORY  
RESET (E.G. FOLLOWING REPAIR OR SERVICING).

- 1 Home button
- 2 ON/OFF button
- 3 Soft button 1
- 4 Soft button 2
- 5 Soft button 3

The **Flocare® Infinity™ III feeding pump** must be set up before first pump start-up after a factory reset. This means setting the language, time and date. The steps that need to be followed to do this are explained below.

- 1 **Turn the pump on** by pressing and holding the *ON/OFF button* on the right-hand side for 2 seconds.
- 2 The **language selection** will then appear on the display. At first pump start-up or after a factory reset, English will appear first in the list.

You can scroll through the language menu and select a language using *soft buttons 1 and 3* on the right-hand side, next to the arrows.

- 2a The following **18 languages** are available for selection: English, Czech, Danish, German, Greek, Spanish, French, Italian, Hungarian, Dutch, Norwegian, Polish, Portuguese, Russian, Finnish, Swedish, Turkish, Arabic.

Press the button on the right-hand side, next to the word “YES”, to confirm the selection.

- 3 After this, it is necessary to **set the time**. To do this, press the button on the right, next to the word “Hours”.

- 3a The current hour can be set by pressing *soft buttons 1* and *3* on the right, next to the “+” and “-” symbols.

To confirm, press the button on the right-hand side, next to the **hours** set (taking this example, next to the number “14”).

- 3b To set the minutes, press the button on the right, next to the word “Minutes”.

The current minute can be set by pressing *soft buttons 1* and *3* on the right, next to the “+” and “-” symbols. To confirm, press the button on the right, next to the minutes set.

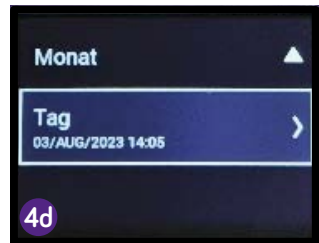
- 4 **Setting the date** comes next.

To do this, press the button on the right, next to the word “Year”.

- 4a The current year can be set by pressing *soft buttons 1* and *3* on the right, next to the “+” and “-” symbols. To confirm, press the button on the right, next to the **year number** set.

- 4b** Next, you will be asked to set the current **month**. As when setting the year number, enter the current month using *soft buttons 1* and *3* on the right, next to the “+” and “-” symbols.
- 4c** To confirm, press the button on the right, next to the month set (taking this example, next to the number “08”).
- 4d** Finally, you must set the current day. As when setting the month, enter the current day using *soft buttons 1* and *3* on the right, next to the “+” and “-” symbols. To confirm, press the button on the right, next to the day set.
- 5** Once the setup is complete, you must again press the *HOME button* at the top left-hand side.
- 5a** This will take you to the home screen, storing the settings that have been entered. You can then switch off the pump by pressing the *ON/OFF button*.

You can find more information on your FloCare® pump and on the full range of feeding tubes, giving sets, pumps and other accessories, as well as training videos, safety information and instructions for use, at <https://nutricia.cc/ernaehrungspumpen>

Subject to change; errors excepted. No liability will be accepted for any printing errors.

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