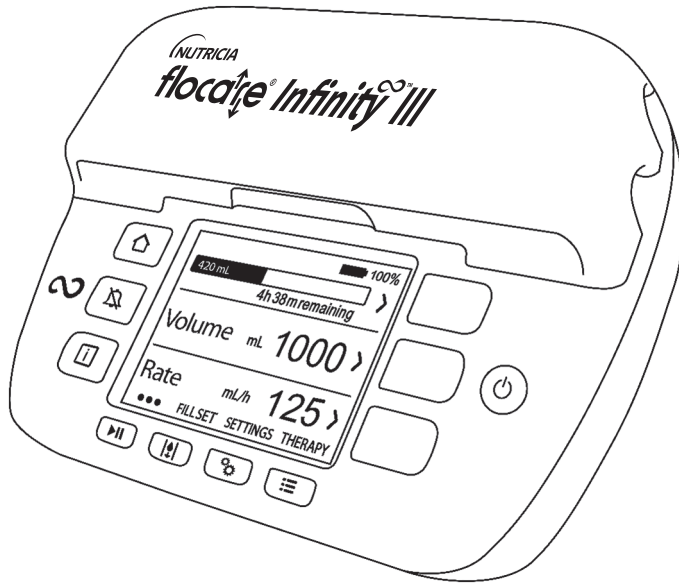
NUTRICIA floca^{re} Infinity™ III

Floca^{re} Infinity™ III pump Instructions for Use

Manufacturer:
Nutricia Medical Devices BV,
Taurusavenue 167,
2132 LS Hoofddorp,
The Netherlands
www.nutriciafloca.com

CE 0344

59512_D / Date of publication November 2023

NUTRICIA CONTACT ADDRESSES

Headquarters	Australia	New Zealand
Nutricia Medical Devices B.V. Taurusavenue 167 2132 LS Hooftdorp, The Netherlands P.O. Box: 75538 Tel nr.: +31 20 456 9000	Nutricia Australia Pty Ltd, The Zenith, Tower B, Level 12, 821 Pacific Highway, Chatswood, NSW, 2067 Australia 1800 060 051, www.nutricia.com.au	Danone Nutricia NZ Ltd, 56-58 Aintree Ave, Airport Oaks, Auckland 2022, New Zealand 0800 636 228, www.nutricia.co.nz
Great Britain	Ireland	Germany
Nutricia Ltd, White Horse Business Park, TROWBRIDGE, Wiltshire, BA14 0XQ, England. Tel: 01225 711 688, www.nutricia.co.uk Patient/Carer Information Number (UK only) Tel: 08457 623 653.	Nutricia Ireland Ltd., Block 1, Deansgrange Business Park, Deansgrange, Co. DUBLIN, Ireland. www.nutricia.ie	DE: Danone Deutschland GmbH, D - 60329 Frankfurt/Main, Deutschland, www.nutricia.de ; AT: Danone Österreich GmbH, A-5412 Puch, Österreich, www.nutricia.at ; CH REP CH: Danone Schweiz AG, CH-8005 Zürich, Schweiz, www.nutricia.ch - Kontakt Danone D-A-CH: 00800 700 500 00
Nederland	Belgium	France
Nutricia Nederland B.V., Antwoordnummer 10230, 2700 VB Zoetermeer-NL, Nutricia Medische Voedingsservice: 0800-022 33 22 (gratis).	N.V. Danone Belux, Werkhuizenkaai 160 - 1000 Brussel, België, Nutricia Medical Careline: 0800 99 486 (gratis) Danone Belux S.A., Quai des Usines 160 - 1000 Bruxelles, Belgique Nutricia Medical Careline: 0800 99 486 (gratuit)	NUTRICIA Nutrition Clinique, 17/19 rue des deux gares, CS 50149, 92565 Rueil-Malmaison Cedex, France. www.nutricia.fr
Italy	Spain - España	France
Distribuito da: Danone Nutricia S.p.A. Società Benefit, via C. Farini 41, 20159 Milano. Per informazioni: 800-822096 www.nutricia.it	Distribuido en España por: Danone Nutricia S.R.L., C/ Torrelaguna 77, 6º plta. CP 28043, Madrid. Tel. Atención al cliente: (+34) 900 211 088. www.nutricia.es ; www.sonda-nutricion.es	NUTRICIA Nutrition Clinique, 17/19 rue des deux gares, CS 50149, 92565 Rueil-Malmaison Cedex. Numéro de téléphone +33 1 49 48 47 00
Portugal	Denmark	Norway
Distribuído em Portugal por: Danone Nutricia Unipessoal, Lda. Apartado 8106, EC Cabo Ruivo, 1802-001 Lisboa, Telef: 214 259 600 Linha Verde: 800 206 799 www.nutricia.pt	Nutricia, c/o Danone A/S, Rørmostevej 2A, 3450 Allerød Tlf. 70210707 www.nutricia.dk	Nutricia, c/o Danone AS, Drammensveien 123, 0277 OSLO Tlf. 23 00 21 00 www.nutricia.no
Sweden	Finland	Poland
Nutricia, c/o Danone AB, Gårdsvägen 14, 16970 Solna. Tel: +46 (0)8-24 15 30. www.nutricia.se	Nutricia, c/o Danone Oy, Siltasaarekatu 8-10, 00530 Helsinki Yhteistyökumppani: Nordic Service Group, Taivaltie 4, 01610 Vantaa, Puh. 010 397 5404.	Nutricia Polska Sp. z o.o., ul. Bobrowiecka 8, 00-728 Warszawa
Czech	Hungary	Greece
Danone a.s., V parku 2294/2, 148 00 Praha 4 - Chodov, infolinka: 800 110 001, www.nutricia.cz	Forgalmazza: Danone Kft., 1134 Budapest, Váci út 35. www.nutricia.hu Infononal: 06 80 223 223	Numil Hellas A.E. 17 ^ο χλμ. Αθηνών Λαμίας & Κολαμάτας 2, 14564 Κηφισιά, Τηλ.: 2106248547, Γραμμή Επικοινωνίας: 800 11 68600

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Middle East

Nutricia Middle East,
Jumeirah Lakes Towers, Cluster W,
Build JBC 5, Unit 3701.
Dubai, United Arab Emirates.

Nutricia CONTACT ADDRESSES.....	2
1. INTRODUCTION	5
2. INTENDED USE	5
3. SAFETY INFORMATION.....	5
3.1. GENERAL SAFETY	5
3.2. WARNINGS AND PRECAUTIONS	5
4. QUICK REFERENCE OVERVIEW	7
5. USER INTERFACE	8
5.1. BUTTONS.....	8
5.1.1.  Home button	8
5.1.2.  Snooze button	8
5.1.3.  Information button	8
5.1.4.  Start/pause button	8
5.1.5.  Fill set button.....	8
5.1.6.  Settings button	8
5.1.7.  Therapy button	8
5.1.8.  Soft buttons.....	9
5.1.9.  On/Off button	9
5.2. PUMP STATUS INDICATOR	9
5.3. DISPLAY	9
5.4. HOME SCREEN	9
6. PUMP SETUP.....	9
6.1. ATTACHING TO THE POLE CLAMP (OPTIONAL).....	9
6.2. CONNECT TO THE AC POWER PLUG [e-IFU].....	9
6.3. SWITCH THE PUMP ON [e-IFU].....	9
6.4. SET THE LANGUAGE	9
6.5. SET THE TIME AND DATE [e-IFU].....	9
6.6. SWITCH THE PUMP OFF	9
7. OPERATING INSTRUCTIONS [e-IFU].....	9
7.1. LOAD THE FLOCARE INFINITY GIVING SET.....	9
7.2. PAUSE / RUNNING MODE [e-IFU].....	10
7.3. FILL THE FLOCARE INFINITY GIVING SET [e-IFU].....	10
7.4. SETTINGS MENU [e-IFU].....	10
7.4.1. Define therapy list	10
7.4.2. Therapy lock	10
7.4.2. Sound [e-IFU].....	10
7.4.2. Therapy completed	10
7.5. PROGRAMMING THE PUMP	10
7.5.1. Program a VOLUME - RATE therapy	10
7.5.2. Program a RATE therapy (not recommended for volume sensitive patients).....	11
7.5.3. Program an INTERMITTENT therapy.....	11
7.5.4. Change the parameters of an active therapy	11
7.6. LAST FEEDING INCOMPLETE.....	11
7.7. INFORMATION & MEMORY FUNCTIONS [e-IFU].....	11
7.7.1. Therapy information	11
7.7.2. 30-day history graph.....	11
7.7.3. Reset progress bar	11
7.8. BATTERY [e-IFU]	11
8. ADVANCED SETTINGS [e-IFU]	11
9. ALARMS, WARNING MESSAGES AND TROUBLE SHOOTING [e-IFU].....	12
10. CLEANING, INSPECTION & SERVICING GUIDELINES [e-IFU].....	12
SERVICING.....	12
ACCESSORIES	12
RECYCLING & DISPOSAL.....	12
11. SPECIFICATIONS [e-IFU]	12
11.1. PERFORMANCE.....	12
11.2. OPERATING.....	12
11.3. STORAGE	13
11.4. EMC.....	13
12. SYMBOLS [e-IFU]	13
13. WARRANTY	13

..... 2 **Instructions for Use**

..... 5
 5 **UK-IE** • This Instructions for Use manual is enclosed with each pump and informs the user of the device of its safe and proper use, its expected performances and of precautions to take. It includes information on how to start up the device as well as basic operations and set up of the pump as to enable users and carers, who received pump training, to set up their therapies themselves.

..... 5 Additional information can be obtained in the 'electronic Instructions for Use' (e-IFU) accessible via www.nutriciaflocare.com. A printed version of this e-IFU can be obtained via the local Nutricia affiliates upon request.

..... 7
 8 For quick reference, this Instructions for Use follows the same structure and build up as the e-IFU. The notion '[e-IFU]' indicates that more information on the topic is available in the electronic IFU.

..... 8 Furthermore, the built-in 'INFORMATION button [I]' of the pump provides help and additional information about the actual pump status at any given time. Amongst others, the INFORMATION button [I] can provide suggestions on the display to solve an alarm or warning condition.

..... 8 **1. INTRODUCTION**

..... 8 The Flocare® Infinity™ III pump is a rotary peristaltic enteral feeding pump. The pump provides a variety of options that allow for customisation for each patient and a flexible approach to enteral feeding therapy.

..... 8 **2. INTENDED USE**

..... 8 The Flocare Infinity III is intended to be used to administer enteral nutrition or hydration fluids in a safe, controlled and intuitive manner into the gastrointestinal tract of a patient who is unable to eat or drink normally.

..... 9 The pump will operate in a clinical environment or home care setting and can be used by trained care givers and patients for bedside as well as for mobile use.

..... 9 **Medical indication**

..... 9 Enteral feeding through enteral pump assisted feeding is indicated for patients at nutritional risk or are malnourished who:

- 9 - cannot meet their nutritional or fluid requirements with normal dietary intake.
- 9 - have a functioning gastrointestinal tract.

..... 9 **Intended target groups**

..... 9 The Flocare Infinity III pump system is intended for adult and/or paediatric patients.

..... 9 **Contra-indications**

..... 9 Do not use:

- 9 - if enteral feeding is contraindicated
- 9 - for Intravenous administration (Parenteral Nutrition)

..... 9 Enteral feeding may not be appropriate for some patients depending on the Individual's diagnosis (e.g., intestinal obstruction).

..... 10 **3. SAFETY INFORMATION**

..... 10 **3.1. GENERAL SAFETY**

- 10 • Read the entire instructions for use before using the Flocare Infinity III enteral feeding pump.
- 10 • The user should be trained in the operation of the Flocare Infinity III and its accessories.
- 10 • The Flocare Infinity III enteral feeding pump shall only be used in combination with Flocare® Infinity™ giving sets. The use of non-Flocare Infinity giving sets will generate unforeseen hazards and will not provide the required functionality and safety. Consult your local Nutricia sales representative for information on available Flocare Infinity giving sets.
- 10 • Replace the Flocare Infinity giving set every 24 hours to maintain delivery accuracy, functionality, allow proper system performance and prevent potential retrograde growth of bacteria. Do not program volume and rate combinations which exceed a 24-hour feed regimen.
- 11 • Enteral pump fed patients should be regularly monitored and supervised. Specific patient groups require consistent and controlled administration of enteral nutrition as well as simultaneous application of medication (e.g. insulin administration). In these cases, regular and frequent checks, as determined by the attending healthcare professional, should be carried out to ensure correct administration of nutrition throughout the therapy period.
- 11 • Strangulation hazard: avoid leaving AC power plug cord and Flocare Infinity giving set tubing where infants or young children can become caught around the neck.
- 11 • Choking hazard: caution should be taken as the Flocare Infinity giving sets contains loose parts which may be dangerous for young children.
- 11 • Using the Flocare Infinity III pump's VOLUME function is recommended to program as good practice.
- 11 • The healthcare giver should verify that the patient can tolerate the programmed volume and flow rate.
- 11 • Do not use the pump in strong magnetic fields. e.g. MRI (Magnetic Resonance Imaging) environments.
- 11 • Notice: any serious incident that has occurred in relation to the device should be reported to Nutricia and the local competent authority.

..... 12 **3.2. WARNINGS AND PRECAUTIONS**

..... 12 **GENERAL WARNINGS**

- 12 • Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- 12 • Medical electrical equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the accompanying documents.
- 12 • Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Flocare Infinity III, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- 13 • Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

- A leaking enteral feeding system might compromise the pump's ability to detect an occlusion. Ensure the enteral feeding system is installed and connected properly. During a feeding therapy, ensure there is no leakage observed within the system. In case of any leakage, tighten the connectors and closure caps and verify whether leakage stopped. In case the leakage still occurs, replace the Flocare Infinity giving set.
- In the event of malfunction of the pump or changes in its performance, please consult your local Sales Representative or care giver for a pump service inspection or replacement as required.
- Plug the pump charger into the mains to charge the battery before commencing enteral feeding.
- Use only approved cleaning agents, using non-approved cleaning agents might harm the pump.
- To avoid electrical shock, switch the pump off and remove the plug from the mains before cleaning the pump.
- Make sure the Infinity AC power adapter is completely dry before plugging into the mains.
- Do not cover the speaker at the back of the pump as to ensure any alarm is readily noticed.

Precautions

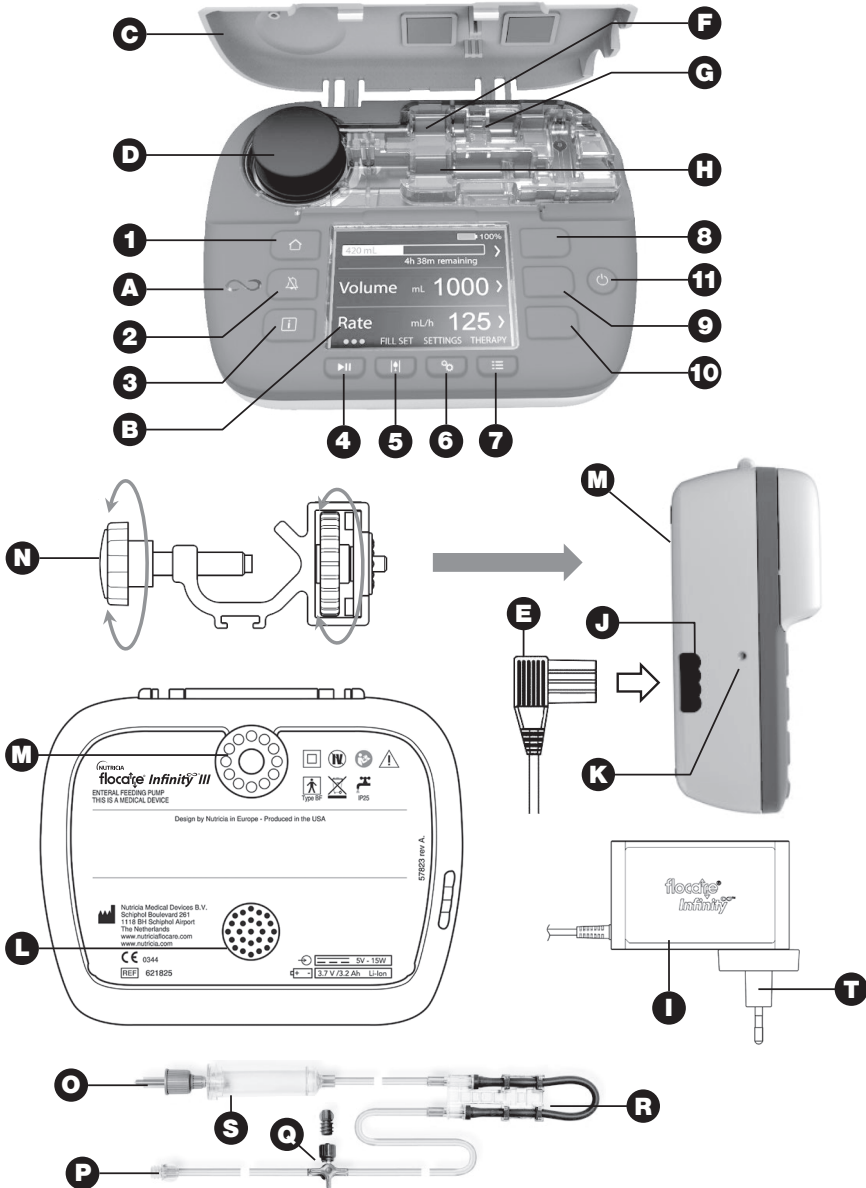
Before using or setting a feeding regimen, check the integrity of the pump for the items below:

- No cracks on the pump housing.
- The power pins of the AC adaptor socket of the pump are of equal length, not bent and not corroded.
- The keypad is not cut or damaged and the text is legible.
- Scratches on the display that affects display legibility.
- Inspect the rotor; verify that the rollers are not damaged, perpendicularly in relation to the rotor and turn freely by hand.
- Verify that the AC power adapter including housing and cord is clean and not damaged.

Caution: Do not use the pump if it is damaged.

- The health care professional is responsible to determine the therapy setting and clinical needs, as well as the appropriate surveillance scheme and its frequency. If the required surveillance scheme cannot be guaranteed by the care giver, then it is to be discussed with the healthcare professional, who can advise the patient an alternative therapy solution.
- For volume and nutrient sensitive patients (e.g. children with metabolic conditions using overnight and/or unmonitored therapy) where a constant nutrition delivery is vital and/or connected with consequential therapies (e.g. an insulin delivery pump); additional surveillance is recommended to be put in place to verify good functioning of the enteral feeding system and programmed therapy.
- The sound level of the Infinity pumps audible alarm is to be set in 'HIGH' if the pump is operated in a noisy environment or the health carer is not nearby the pump, such as at night, to ensure the alarm is readily noticed when an alarm is activated. The intended operator position is in front of the pump.
- Nutricia recommends using ready to feed tube nutrition as this is developed for the intended delivery specifications of the pump.
- Any nutrition added into the enteral feeding line must be of a homogeneous nature and should remain homogeneous during the feeding regimen.
- Use only feeding solutions prescribed by the responsible clinician, registered dietitian, registered nurse or other licensed practitioner.
- Before commencing any enteral feeding therapy, proper consideration is given to the feed appropriateness for tube feeding (thickness, homogeneity, selection of nasogastric tube Ch size, etc.), as to avoid occlusion or any other unexpected pump system behavior. The condition and appropriateness of the nutrition can be addressed with the healthcare professional, who knows the clinical situation and needs of the patient and can best guide the patient in the selection of appropriate feed.
- Non-liquid food ingredients and mixed or reconstituted foods may tend to occlude the flow within the feeding tube and Flocare Infinity giving set. Clinicians and caregivers should carefully monitor the pump during use with these types of feeds.
- Users must ensure that any substrate used for any mixed or reconstituted powdered nutrition is indicated as suitable for enteral tube feeding delivery (refer to instructions for use on nutrition / substrate added).
- Mixed or reconstituted nutrition may have foam. If using this type of nutrition, allow it to sit for some time until the foam has been dissolved before pouring it into a feeding reservoir. This will reduce the chance of an 'AIR in giving set' alarm.
- If other forms than a standard, ready to feed formula (i.e. commercially available pre-packed or commercially prepared feeding solutions) are prescribed as enteral tube feed, it should be done with caution and aligned with the healthcare professional. Feeds supplied as tube feeds must be of a homogenous nature and should remain homogeneous during the feeding regimen as bigger particles or high viscous foods might result in blockages in the feeding line. It is noted that for instance 'home-made' – mixed tube feeds might give rise to issues when given as an enteral tube feed as the mixture will not stay in emulsion for a longer period
- In using any mixed, reconstituted powdered nutrition or breast milk, we advise users to ensure the nutrition is prepared and dissolved properly as to avoid particles, foam accumulation or films in the Flocare Infinity giving set potentially hampering pump performance and a normal feeding regimen.
- If patients are advised by their healthcare professional to use a mixture of feeds, the feed must be given when there is direct supervision (e.g. parents) to ascertain the feeding therapy is running normally and to ensure proper actions are taken in case there is a pump failure or unexpected behaviors caused by the mix.
- Do not modify this equipment. Modification of this device or its accessories can result in physical hazards including delayed therapy, over delivery, under delivery, electrocution and fire. These hazards could result in patient injury or death.
- Do not use the Infinity alarm systems to trigger actions related to secondary (electrical) medical devices (e.g. a volumetric or syringe pump).
- Do not use pump functions for any other purpose than described in this manual. e.g. FILL SET function to overcome an occlusion or the AIR in giving set alarm to indicate end of therapy.

4. QUICK REFERENCE OVERVIEW

A	Pump status indicator	H	Occlusion OUT sensor	O	ENPlus® connector
B	Colour display	I	AC power plug	P	ENFit® connector
C	Door	J	AC power socket	Q	Medication port
D	Pump rotor	K	AC charging light	R	Cassette
E	Charger plug	L	Speaker	S	Drip chamber
F	Occlusion IN sensor	M	Pole clamp interface	T	Detachable AC plug
G	Air sensor	N	Pole clamp		

5. USER INTERFACE

5.1. BUTTONS

1	 Home button	5	 Fill set button	9	SOFT button 2
2	 Snooze button	6	 Settings button	10	SOFT button 3
3	 Information button	7	 Therapy button	11	 ON/OFF button
4	 Start/pause button	8	SOFT button 1		

5.1.1. Home button

The HOME [] button navigates back to the home screen or dashboard, in 1 touch from whichever screen you are in.

Additionally, the HOME [] button acknowledges an alarm or warning.

Note: the HOME [] button does not act as a back button to move one level up in a menu.

5.1.2. Snooze button

The alarm SNOOZE [] button silences the audio alarm signal for 2 minutes when the pump is in an alarm state.

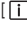
This gives you the opportunity to investigate and addresses the indicated alarm without being distracted by the audio alarm signal.

Note: the alarm remains visible. The  symbol on the display indicates that the alarm sound is snoozed.

5.1.3. Information button

The INFORMATION [] button (i-button) provides help and additional information about the actual pump state at any given time.

Examples:

- In a running mode, the information button provides information about the status of the therapy.
- In pause mode, the information button will give recommendations how to program a therapy depending on your selected therapy.
- In an alarm state, the information button [] provides recommendations on how to solve the indicated alarm by providing potential solutions.

5.1.4. Start/pause button

The START/PAUSE [] button starts or pauses a programmed therapy.

5.1.5. Fill set button

The FILL SET [] button primes the Flocare Infinity giving set to ensure that air within the system has been purged.

5.1.6. Settings button

The SETTINGS [] button provides access to the pump settings.

See chapter 7.4 'Settings menu' for full button functionality.

5.1.7. Therapy button

With the THERAPY [] button, you can select your desired therapy.

See also chapter 7.4. SETTINGS MENU

Reset patient data is designed to erase all collected patient data (non-personal)

The following data will reset: *Therapy parameters, Progress bar, Patient history data, Personalised therapies, Define therapy list, Sound settings and Display settings.*

5.1.8. Soft buttons

The soft buttons relate to the function on the screen which has an action indicated by the “<, >, ▲, ▼, ◀, ▶, +, -” symbol.

Note: the SOFT buttons are only active when accompanied with the above-mentioned symbols, if no symbol appears, the button is inactive.

5.1.9. On/Off button

The ON/OFF button switches the pump ON and OFF.

Note: to avoid unintended button presses, the ON/OFF button needs to be pressed for 2 seconds for both switching the pump ON or OFF.

5.2. PUMP STATUS INDICATOR

The pump status indicator provides a visual indication of the actual pump status.

- A green light indicates that the pump is in running mode and working normally.
- An orange light indicates that the pump is in pause mode and waits for user input.
- A red light indicates an alarm state; immediate action from the user is required.

5.3. DISPLAY

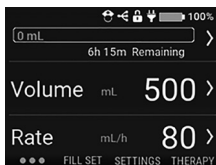
The Flocare Infinity III will perform a display test during each pump start. Carefully check for any sections or pixels that do not appear during the display test.

Caution: In case of any missing sections or pixels, the pump should not be used and to be send to an authorised service centre for inspection.

5.4. HOME SCREEN

The home screen lay-out corresponds with your selected therapy VOLUME / RATE, RATE or INTERMITTENT.

The grey lines separate the panels in PAUSE mode, and are a visual indicator that the navigation and selection buttons are active.

6. PUMP SETUP *[see eIFU]*

6.1. ATTACHING TO THE POLE CLAMP (OPTIONAL)

1. Connect the Flocare Infinity III pole clamp (N) to a pole stand by rotating the black star shaped bolt clockwise.
2. Connect the Flocare Infinity III to the pole clamp interface (M) by rotating the grey wheel shaped disk clockwise until the pole clamp is fully connected to the pump without any clearance.

6.2. CONNECT TO THE AC POWER PLUG

Connect the AC power adapter by gently pressing the charger plug (E) into the pump (J).

The charger plug is designed to fit in only one way. When the pump is connected to the mains, the blue AC charging light (K) should be on and the plug icon to be appeared next to the battery icon.

Note: The Flocare Infinity III is equipped with Lithium/Ion battery technology and needs to be fully charged for a period of 6 hours prior to first use.

6.3. SWITCH THE PUMP ON

Turn the pump ON by pressing and holding the ON/OFF button for approximately 2 seconds until the start-up sound can be heard. The pump will perform a self-test and inform the user when the device is ready to use, the home screen will be shown after the self-test.

Note: When a previous therapy has not been administered completely before the pump was turned OFF, the pump will show the warning message 'Last feeding incomplete'.

6.4. SET THE LANGUAGE

Set the language by pressing the SETTINGS  button, navigate to LANGUAGE and select your preferred language.

Note: at first pump start up or after a factory reset, the pump will guide you automatically through the language selection.

6.5. SET THE TIME AND DATE

Set the date and time by pressing the SETTINGS  button, navigate to TIME AND DATE and set these to your local time zone. The pump will not change to daylight savings automatically nor will it change to other time zones during travelling.

Note: at first pump start up or after a factory reset, the pump will guide you automatically through the date and time setting.

6.6. SWITCH THE PUMP OFF

Switch the pump OFF by pressing and holding the ON/OFF button for approximately 2 seconds until you hear the turn-off sound

7. OPERATING INSTRUCTIONS *[see eIFU]*

7.1. LOAD THE FLOCARE INFINITY GIVING SET

Follow the instructions for use of the Flocare Infinity giving set, for setting up the enteral feeding system.

1. Open the door

2. Position the purple looped section of the cassette (R) around the pump rotor (D)
3. Gently stretch (do not over stretch) and manually press the giving set into position by pressing the cassette downwards into the pump top housing. The cassette is designed to fit into the pump one way
4. Close the pump door
5. Pump is now ready to be programmed

7.2. PAUSE / RUNNING MODE

The Flocare Infinity III is either in pause mode waiting for user instructions or is in running mode delivering a therapy to a patient. The pause or running modes are controlled by the START/PAUSE [▶||] button.

Note: when the pump has been untouched for 3 minutes or more in PAUSE mode, the pump will display the 'Pump needs attention' alarm message.

7.3. FILL THE FLOCARE INFINITY GIVING SET

The FILL SET function is used to prime a (new) Flocare Infinity giving set with nutrition or water and needs to be commenced before starting a feeding therapy. Ensure the Flocare Infinity giving set is disconnected from the patient feeding tube. Prime the set by press and hold the FILL SET [!@] button for 2 seconds to activate the FILL SET function and release the FILL SET [!@] button as soon as the pump generates a beep and automatically starts. The FILL SET can be manually stopped by pressing STOP in case the giving set is over filled.

7.4. SETTINGS MENU

7.4.1. Define therapy list

DEFINE THERAPY LIST controls the list of available therapies that are visible when the THERAPY [≡:] button is pressed. Therapies that are currently enabled are marked with a green switch symbol.

7.4.2. Therapy lock

The THERAPY LOCK will lock an active therapy to avoid therapy tampering. It is not possible to select a new therapy or reprogram the active therapy when the THERAPY LOCK is activated, however it is possible to restart a locked therapy. Clearing the progress bar is still possible. A therapy lock can be activated in PAUSE mode only by pressing SETTINGS THERAPY LOCK and a valid PIN code must be entered. When a therapy is locked, a lock symbol appears in the status bar. Unlocking a therapy is done via the same procedure.

7.4.3. Sound

The Flocare Infinity III supports 3 AUDIO VOLUMES, 'Low, Medium and High' and can be set by the user. For safety reasons the alarm volume is automatically escalated to "High" in both settings 'Low' and 'Medium' after 3 min. This safety feature cannot be turned off.

7.4.4. Therapy completed

Notice: This menu option is only available from software version 1.07.0088 onwards. The Flocare Infinity III can give a notification once the delivery of the programmed therapy is completed. This VOLUME DELIVERED alarm supports three different user-configurable audio options. 'CONTINUOUS' is activated by default and the pump will generate a visual warning and a recurring audio alarm. 'ONLY ONCE' A single audio alarm once the delivery of the programmed therapy is completed. 'OFF' If no notification on programmed therapy is required, the VOLUME DELIVERED alarm can be switched off.

Caution: consult your care giver before disabling the VOLUME DELIVERED alarm.

7.5. PROGRAMMING THE PUMP

Caution: Before programming the pump, always ensure that your selected therapy and its parameters are correct. In case of any doubts, consult your care giver.

Therapies are programmed via the THERAPY [≡:] button. The Flocare Infinity III supports the following therapies:

Volume – Rate	Administering of a programmed VOLUME (mL) with a programmed RATE (mL/h).
Rate	Administering with a programmed RATE (mL/h) but without a programmed VOLUME (mL)
Intermittent	Administering of a programmed VOLUME (mL) divided over different portions with a programmed RATE (mL/h) with programmed PAUSE (interval) times.

IMPORTANT:

Therapies RATE and INTERMITTENT are disabled by default, if a desired therapy does not appear when the THERAPY button is pressed, you can enable them by selecting: SETTINGS → DEFINE THERAPY LIST

7.5.1. Program a VOLUME - RATE therapy

- From the home screen press the THERAPY [≡:] button
- Select VOLUME – RATE by pressing the associated soft button.
- Press the VOLUME soft button
- Change the Volume using the '+' and '-' buttons, and press the '>' button to accept the value.
- Press the RATE soft button
- Change the Rate using the '+' and '-' buttons, and press the '>' button to accept the value.
- Check your total therapy time underneath the VOLUME or RATE panel for additional safety.
- Press the START/PAUSE [▶||] button to start your VOLUME-RATE therapy.

7.5.2. Program a RATE therapy (not recommended for volume sensitive patients)

- From the home screen press the THERAPY [≡] button
- Enter the RATE soft button
- Change the Rate using the '+' and '-' buttons, and press the '>' button to accept the value.
- Press the START/PAUSE [▶||] button to start your RATE therapy.

7.5.3. Program an INTERMITTENT therapy

The INTERMITTENT therapy has a step-by-step approach where all parameters need to be entered and validated prior to return to the home screen. The Flocare Infinity III shows resulting parameters based on the user input for safety and convenience reasons.

Note: after each step, the Flocare Infinity III guides you to the next programming step.

- From the home screen press the THERAPY [≡] button
- Select INTERMITTENT by pressing the associated soft button.
- Press the TOTAL VOLUME soft button
- Change the Volume using the '+' and '-' buttons, and press the '>' button to accept the value.
- Press the PORTIONS soft button
- Change the required Portions using the '+' and '-' buttons, and press the '>' button to accept the value.
- Press the RATE soft button
- Change the Rate using the '+' and '-' buttons, and press the '>' button to accept the value.
- Press the PAUSE soft button
- Change the required Pause time using the '+' and '-' buttons, and press the '>' button to accept the value.
- Scroll up and down with soft button 1 and 3 through all programmed settings and verify the input
- Scroll to CONFIRM and validate the input by pressing soft button 2.
- Press the START/PAUSE [▶||] button to start your INTERMITTENT therapy.

Note: When the pump is in the NEXT PORTION mode, the home screen will differ, indicating and counting down the time until the next portion. The running indicator will remain active during the count down.

7.5.4. Change the parameters of an active therapy

Changes to an active therapy are possible regardless whether a manual programmed or a personalised therapy is active. To change an active therapy, make sure the pump is in PAUSE mode.

Volume – Rate VOLUME and RATE parameters can be modified

Rate RATE parameters can be modified.

Intermittent RATE parameter can be modified.

TOTAL VOLUME cannot be modified.

PAUSE time can be modified only when the pump is in NEXT DELIVERY mode.

7.6. LAST FEEDING INCOMPLETE

When a therapy has not been administered completely whilst the pump has been turned OFF, the pump will show the warning message: 'Last feeding incomplete'. You either have the option to RESTART the active therapy where the progress bar will be reset or you can RESUME the delivery of the active therapy.

7.7. INFORMATION & MEMORY FUNCTIONS

7.7.1. Therapy information

When a 'VOLUME – RATE' or 'RATE' therapy has been programmed, the Flocare Infinity III provides useful information concerning your active therapy when the THERAPY INFORMATION button is pressed. This function can be accessed from the HOME SCREEN, SOFT BUTTON 1, THERAPY INFORMATION

7.7.2. 30-day history graph

The Flocare Infinity III stores all administered therapies for a maximum period of 30 days. The daily administered volume can be accessed by pressing SOFT BUTTON 1 from the home screen and pressing HISTORY. With the arrow buttons, you can navigate to earlier or later days. By pressing the + magnifying symbol you can zoom in on your selected day by showing a table of individual delivered therapies

7.7.3. Reset progress bar

The Progress bar is automatically cleared after completion of each therapy where a VOLUME was programmed. When there is a need to manually clear the Progress bar, navigate from the HOME screen to SOFT BUTTON 1 → RESET PROGRESS BAR

7.8. BATTERY

The Flocare Infinity III is equipped with Lithium/Ion battery technology and needs to be fully charged for a period of 6 hours prior to or during its first use, this will maximise the battery life time. The battery microprocessor continuously monitors the battery state of health and the pump displays in the 'About this pump' screen when the battery is depleted for 70% or more and needs replacement.

Note: The battery capacity is an estimate. If you are unsure that enough battery capacity remains for your intended therapy, connect to the AC power adapter until recharged.

8. ADVANCED SETTINGS

Advanced settings menu is intended for qualified caregivers or qualified technicians/ BMET only.

9. ALARMS, WARNING MESSAGES AND TROUBLE SHOOTING [see eIFU]

The FloCare Infinity III is equipped with several sensors and safety features linked to audible and visible alarms and warning messages on the display. In case any of the alarms occur, the pump will generate an audible and visual alarm and stops administering. The back light of the display automatically switches on. Exception to this is the low battery alarm, in this situation the pump continues working. Infinity alarm conditions are classified as low priority indicating that operator awareness is required and future action is needed. If an alarm or warning message appears on the display:

- The HOME [] button acknowledges an alarm or warning state and dismisses the alarm or warning message.
- The SNOOZE [] button snoozes the alarm or warning audio signal for 2 minutes whilst the alarm or warning message remains on the display. A suppressed audio icon will appear on the display indicating that the audio signal has been snoozed.
- The INFORMATION button [] can provide suggestions on the display to solve an alarm or warning condition.

Note: during an alarm or warning message, all other buttons are disabled. The HOME [] button is the only button that acknowledges an alarm or warning message.

10. CLEANING, INSPECTION & SERVICING GUIDELINES [see eIFU]

For optimum and safe pump operation, the pump should be kept clean and free from debris. It is recommended to clean the pump with a wet cloth after every completed feeding regimen.

Disconnect the pump from the mains before cleaning. To avoid electrical shock, never clean the pump with AC power adapter plugged into an outlet or when pump is on.

Do not immerse the pump in water.

Do not use aggressive cleaners or disinfectants such as acetone, benzene or iodine.

Note: the pump is not intended to be cleaned using a dishwasher.

Do not sterilize the pump, e.g. by irradiation, EtO, steam or in an autoclave. It is recommended to clean the pump regularly under (running) warm tap water with standard washing up liquid, a non-abrasive sponge, brush or soft cloth. Clean and remove any nutritional residue from the pump paying particular attention to the sensor area on the top housing, the rotor including its rollers and in the corners of the keypad and screen. Then dry the pump with a clean cloth.

After cleaning, the pump can be disinfected using a mild, diluted disinfectant or cleaning solution and damp cloth. Consult your Nutricia representative or health care provider/ pump service provider for approved disinfection agents. In case any effects are observed in using the agent or in case of any doubt in approved cleaning agent usage, potential negative effect of cleaning agent used or concerns/ complaints, stop using the agent, consult the products instructions for use or contact your local Sales Representative for further information and verification.

SERVICING

There are no user-serviceable parts, servicing activities are to be performed at an approved service centre or a Nutricia qualified Technician / BMET. An inspection of the pump by an authorised service centre or a trained and qualified technician / BMET is recommended in the following cases:

- Switching between patients
- In case of any change in performance characteristics or pump alarm messages/warnings
- In case the pump has been dropped

The pump does not require preventive maintenance, routine servicing, or calibration under normal conditions of use in cases where proper cleaning and inspection protocols are adhered to, for a period of 5 years.

ACCESSORIES

Dedicated carry bags and pump stands, as well as Infinity giving sets and an Infinity Nurse call (cable length 5m) and Infinity PDMS data cable (cable length 3m) are available for the FloCare Infinity III pump. Please contact your local Nutricia sales representative for availability and ordering information of the respective parts.

RECYCLING & DISPOSAL

The pump and its electrical accessories (AC power plug, PDMS cable, Nurse Call cable), should be disposed according to local standards and regulations governing the disposal of electronic waste (e-waste). Other accessories can be disposed of or recycled as standard non-hazardous waste.

Note: In case of pump scrapping, always notify your Nutricia sales representative of the pump serial number

11. SPECIFICATIONS [see eIFU]

11.1. PERFORMANCE

Volumetric accuracy (1)	±5%
Battery life (batteries not older than 2 years)	24h @ 125mL/h, 12h @ 250mL/h
Air bubble alarm	> 1.5 mL of air in giving set after the pump set.
Upstream occlusion pressure	-34 kPa ± 21 kPa for 5 seconds;
Downstream occlusion pressure	84 kPa ± 21 kPa for 30 seconds/ 103 kPa ± 21kPa for 5 sec

11.2. OPERATING

Atmospheric pressure	65 – 106 kPa
Temperature	5° – 40°C
Humidity	15 – 95%, non-condensing

11.3. STORAGE

Atmospheric pressure	50 – 106 kPa
Temperature	-20° – 60°C
Humidity	10 – 95%, non-condensing
Shelf life	None. A normal expected pump (technical) use life is 5 years. The battery life degrades over time and might be subjected to earlier replacement. The expected service life of the battery is 2 to 5 years, depending on usage and number of charging cycles.

Note: Environmental conditions listed apply before and after the pump has been removed from the packaging.

Note: Time to recover from maximum and minimum storage temperature to operating temperature is ~30 min.

11.4. EMC

Electromagnetic emissions may affect the operation of any electronic medical device, including enteral feeding pumps. The Flocare Infinity III will not be affected by electromagnetic emissions in most environments. However, some electromagnetic fields produced by personal communication equipment, household appliances, or occupational tools may cause electromagnetic interference (EMI) which can affect the pump's essential performance.

Possible sources of electromagnetic interference with electronic medical devices include, but are not limited to: cellular phones, cordless telephones, microwave ovens, anti-theft/security systems, blenders, and high-powered tools.

Check the pump regularly if operating near sources of electromagnetic emissions.

The Flocare Infinity III is designed in accordance with EN 60601-1-2, EN 60601-1-4 for electromagnetic emissions and immunity. (Refer to APPENDIX A: GUIDANCE AND MANUFACTURER'S DECLARATION–ELECTROMAGNETIC COMPATIBILITY in the eFU). The use of other accessories, adaptors and cables than listed within this manual may result in increased emissions or decreased immunity of the equipment of the Flocare Infinity III pump.

12. SYMBOLS

-  Manufacturer  Consult instructions for use
- SN** Serial number  Not for IV use (enteral use only)
-  Mechanical and fluid ingress  CE mark and Notified Body identification number
IP25

13. WARRANTY

All service and repair activities should only be carried out by an approved service centre, authorised repair centre or a Nutricia qualified Technician /BMET. The warranty is for a period of 36 months for customers from the date of delivery. Warranty includes the Flocare Infinity III pump, pole clamp, and AC power plug. Warranty covers replacement, within the period for normal wear and tear and not from misuse.

Limitations on warranty

Warranty will be void if the product has been misused, tampered with or the performance specifications have been altered. Warranty will be void if the product is used outside of the recommendations contained with this IFU. Any repairs conducted by a non-authorised centre will void the warranty.

Manufacturer:
 Nutricia Medical Devices BV,
 Taurusavenue 167,
 2132 LS Hoofddorp,
 The Netherlands,
 www.nutriciaflocare.com

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